



③ Returning Sample



1. Place the saliva tube inside of the plastic biohazard bag. Do not remove the absorbant material from the bag.

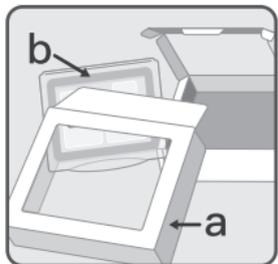

2. Discard the inner package insert (a) and the plastic clamshell (b).

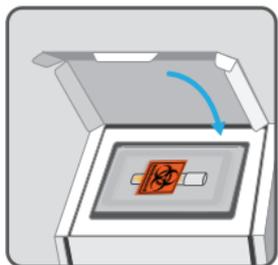

3. Place the plastic biohazard bag containing the saliva tube in the original product packaging and carefully close it.



4. Affix the prepaid USPS return label onto the white poly mailing bag.



5. Place the product packaging and test requisition form into the white poly mailing bag and seal tightly.



6. Drop off the package at your nearest post office within 24 hours.



Saliva Collection Kit Instructions

CellMax
DNA Genetic
Cancer Risk Test

① Before Returning Sample

1. Obtain physician authorization

- Method A (Independent physician network)

Fill out the test requisition form included in your saliva collection kit **except for the Physician Authorization section**. Return your saliva sample along with the test requisition form. CellMax Life will arrange for an independent physician to review your test requisition form and authorize testing if approved.

- Method B (Using own physician)

Make an appointment with your physician to discuss genetic testing. Have your physician completely fill out and sign the test requisition form included in your saliva collection kit. Be sure to include the physician completed test requisition form in the return saliva sample shipment.


CellMax Life must have physician authorization to process any saliva samples.

2. Carefully read the saliva collection kit instructions.


DO NOT eat, drink, smoke, chew gum, floss, or brush teeth 30 minutes prior to saliva collection. 10 minutes prior to saliva collection, gently rinse mouth with water.

② User Instructions


1. Rinse mouth with water. Wait 10 minutes, then spit into Collection Funnel until the amount of liquid saliva (not bubbles) in the Collection Tube reaches the 2 mL line. If you notice excess bubbles, eliminate them by gently tapping the tube on a hard surface. Discard or recycle the Collection Funnel after collection.

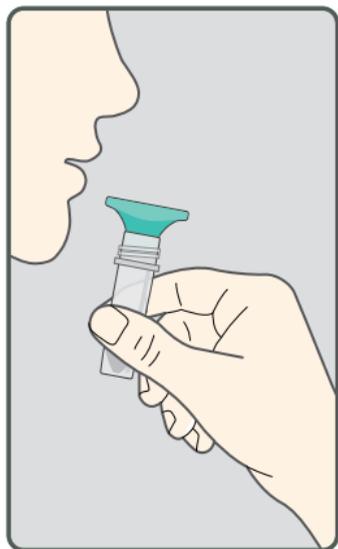

2. Twist open the tip of the Preservative Ampoule. Squeeze the Preservative Ampoule to empty the entire contents into the Collection Tube. Discard or recycle the empty Preservative Ampoule. Screw on Collection Tube Cap tightly to close. Shake the Collection Tube well for 10 seconds to mix saliva sample and preservative.

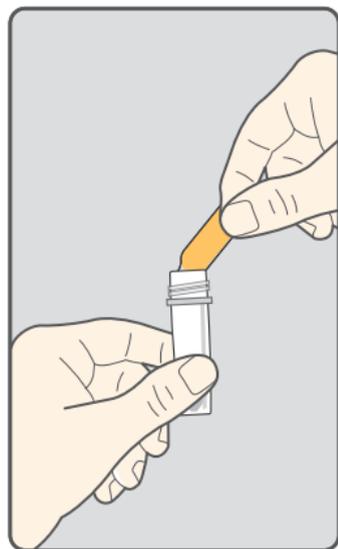

3. Write your name and date of birth on the included label, exactly as how it appears in your test requisition form. Attach the completed label on the Collection Tube.
Do not cover up the barcode.
The sample is now preserved and ready for shipping. Follow the instructions on returning sample to CellMax.

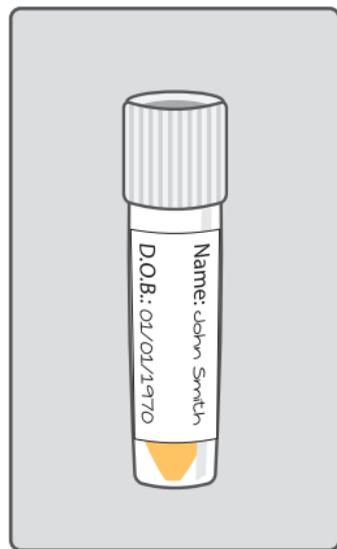

Kit Contents

